

WOUND CONTACT LAYER

DERMANET®

DERMANET®



- Non-adherent
- Shields fragile granulation tissue
- Ideal for lining
- Helps reduce trauma

 DeRoyal®



WOUND CONTACT LAYER

DERMANET®

FEATURES & BENEFITS

- An inert, non-adherent wound contact layer dressing
- Place directly on wound to protect wound surface during healing
- Shields fragile granulation tissue from disturbance caused by secondary dressings
- May be used dry or coated with topical ointment
- Ideal for lining deep wounds before the application of packing materials or under gauze
- Helps reduce trauma during wound dressing removal
- Available in sheets or rolls
- Use as primary dressing


INDICATIONS

- Burns
- Chronic wounds

Product #	Tape	Qty/Bx	Qty/Cs
46-145	3" x 3"	25	50
46-143	5" x 4"	25	50
46-142	8" x 10"	25	50
46-141	20" x 15"	25	50
46-140	24" x 36"	n/a	50
46-144	6" x 72" Roll	n/a	25

TO PLACE AN ORDER OR FOR MORE INFORMATION
PLEASE CALL: **800-251-9864**
888.938.7828 (WITHIN USA)
865.938.7828 (OUTSIDE USA)
800.543.2182 (FAX)

BURNS

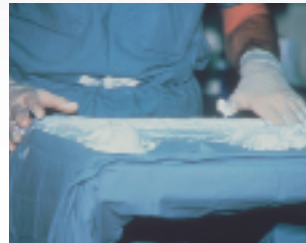

Apply topical ointment to Dermanet.



Wrap around burn site.



Apply chosen dressing.



Removes easily without sticking.

CHRONIC WOUNDS



Line cavity. Extend Dermanet beyond wound edge.



Pack wound.



Cover with secondary dressing.



To remove, lift edges of Dermanet.



200 DeBusk Lane
Powell, Tennessee 37849 U.S.A.
www.deroyal.com

Reprint #0-581 • ©2005 DeRoyal.
All Rights Reserved.
Rev. 8/2005